

## Applying the ProRen FLO Condom

First, wash the penis using soap and water. Rinse and dry the penis carefully.

Removing hair from the base of the penis to keep it from being caught in the condom.


Inspect the penis to make sure it does not have any broken or reddened skin.

Wash, rinse, and dry your hands.

Hold the penis at a 90 degree angle to the body.

Place the ProRen FLO at the end of the penis, leaving about a half-inch between the tip of the penis and the FLO Connector.

Gently roll the condom back over the penis.

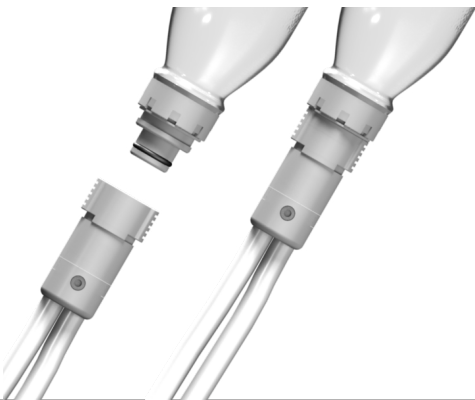

Connect the hose to the FLO Connector. There will be an audible snap when it is locked in place.

To remove the connector, simply squeeze the hose connector, hold the FLO Connector, and pull them apart.

## ProRen FLO

Connect the other end of the hose to the control unit on the collection bag.

The system is fully automatic. When the sensors in the FLO detect urine, the pump will activate and pull the urine away from the skin and into the bag.



The hose easily detaches, allowing the use of intermittent catheters without removing the condom.

