

# Body Type and Harness Sizing

HARNESS SIZE	XS/S	M/L	XL Stabilization Kit REQUIRED	XXL/3X Stabilization Kit REQUIRED
WAIST SIZE (in Inches)	24-30	32-38	40-42	44-50

Type  
**A**



Ideal

Type  
**B**



Ideal

Type  
**\*C**




Marginal

Type  
**\*D**



Marginal

Type  
**\*\*D**

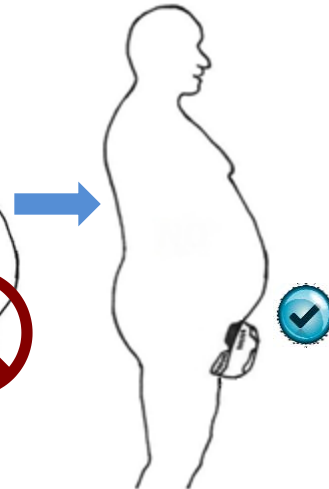

Ideal with stabilization kit.

The diagrams on the left depict a range of body types from ideal A & B to marginal C & D for the proper fit and performance of the Male Cup.

\* Omni recommends that body types C & D utilize the stabilization kit to ensure the Cup does not hang down into a horizontal position. For the Male Cup to function properly it should be held firmly against the body in a near vertical position.

\*\* Most users with body types C & D can use the stabilization kit to fit the Male Cup on their body properly. (Cup must be near Vertical)

\*\*The URINCare® System is not indicated for patients with less than 1 1/2 inches in length or have a retracted penis.